

BR-2X sleep apnoea breathing therapy equipment

Fast Operating Instruction

1.Button Function Description

Title	Function Description
Power Standby	A short button pressed can start ventilation. A long button pressed can stop ventilation.
SEL	Selection and confirmation of menu items
Enter	Return to the upper menu or return to the main interface.
“+” “-”	It is used for the movement of the cursor and the addition and subtraction of the setting item.



2.Equipment Mounting

a) Connect the end of the airway pipe to the assembled mask.	<p>The diagram shows a nasal mask with a black airway pipe attached to it. A black corrugated air tube is shown being connected to the end of the airway pipe. Labels 'Mask' and 'Air tube' point to their respective parts.</p>
b) Connect the other end of airway pipe to the outlet of humidifier.	<p>The diagram shows the humidifier section of the equipment. A black corrugated air tube is shown being inserted into the outlet of the humidifier. A label 'Air tube' points to the tube.</p>

c) Connect the plug of the power adapter to the equipment.





3. Start or stop a measurement

A short Power Standby button pressed can start ventilation. A long button pressed can stop ventilation.

4. Setting of Operating Parameters

Plug in the power, the equipment will directly enter the main page interface (Standby status)

A short Power Standby button pressed can start ventilation. A long button pressed can stop ventilation.

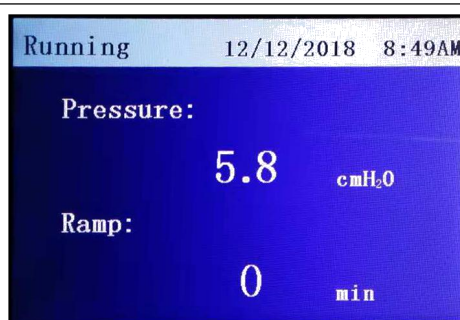
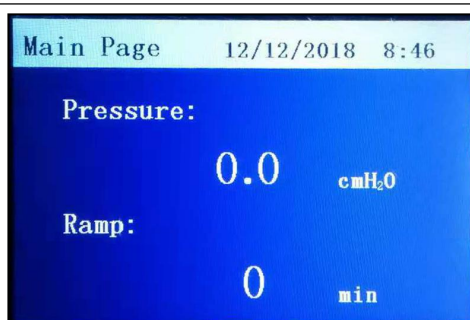


In standby status, press [Power standby] and [Enter] buttons for more than 3 seconds to enter the setting interface of the therapy equipment.

In ventilation mode, press [Power standby] and [enter] buttons for more than 3 seconds to enter the parameter setting interface of current operating mode.

Note:

You can only set the operating mode when you enter the setting interface in standby mode, and you can't change the operating mode in the running mode.

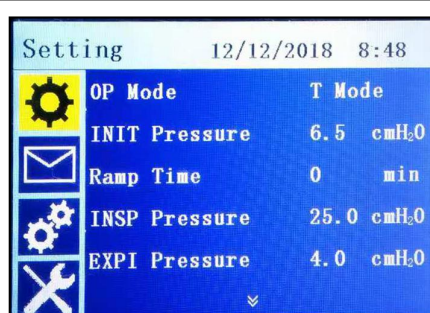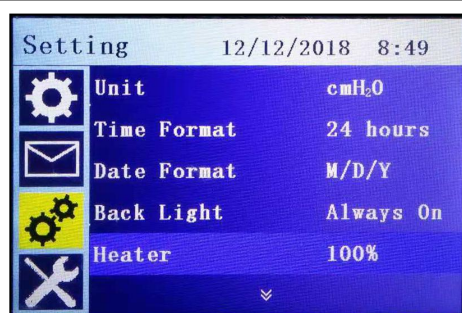

The equipment can operate up to 6 modes, They are CPAP, Auto CPAP, S Mode, Auto S Mode, T mode and S/T Mode respectively.

Note:

Before using the equipment, you need to read the full manual carefully to ensure that you can operate the equipment correctly